

# CYPATH™ LUNG SPUTUM SAMPLE COLLECTION INSTRUCTIONS



NEED HELP?

CALL **855 MYLUNGS** TO SPEAK WITH YOUR PATIENT COACH

# GET READY

REMOVE COLLECTION CUP

SAVE YOUR KIT BOX

READ INSTRUCTIONS

MAKE SURE YOUR NAME IS WRITTEN ON CUP



NEED HELP?

CALL **855 MYLUNGS** TO SPEAK WITH YOUR PATIENT COACH

# **FREEZE THE COLD PACK**

KEEP KIT BOX AND ALL ITS CONTENTS IN SAFE PLACE  
TO USE WHEN YOU RETURN YOUR SAMPLE



*NEED HELP?*

*CALL **855 MYLUNGS** TO SPEAK WITH YOUR PATIENT COACH*

# PREPARE TO COLLECT YOUR SAMPLE

REMOVE THE LID FROM THE COLLECTION CUP

PLACE CUP AND ASSIST DEVICE IN YOUR BATHROOM  
WHERE YOU WILL COLLECT YOUR SAMPLE



*NEED HELP?*

*CALL 855 MYLUNGS TO SPEAK WITH YOUR PATIENT COACH*

# PREPARE YOUR SAMPLE

NEED HELP?

CALL **855 MYLUNGS** TO SPEAK WITH YOUR PATIENT COACH

# USE THE ASSIST DEVICE TO OPEN YOUR AIRWAYS

BREATHE DEEPLY 10 TIMES WITHOUT DEVICE

BREATHE DEEPLY 10 TIMES THROUGH THE DEVICE IN YOUR MOUTH

REMOVE THE ASSIST DEVICE FROM YOUR MOUTH



*NEED HELP?*

*CALL 855 MYLUNGS TO SPEAK WITH YOUR PATIENT COACH*

# COUGH YOUR SAMPLE INTO THE COLLECTION CUP

"HUFF" COUGH 3 TO 5 TIMES TO HELP MUCUS COME UP FROM YOUR LUNGS

EXPEL MUCUS ONLY INTO THE COLLECTION CUP

REPEAT, OR TRY AGAIN LATER, IF NEEDED



*NEED HELP?*

*CALL 855 MYLUNGS TO SPEAK WITH YOUR PATIENT COACH*

# REFRIGERATE YOUR SAMPLE IN THE CUP BETWEEN COLLECTION

SECURE THE LID ON YOUR COLLECTION CUP



Place your collection cup in the refrigerator after each collection.

**DO NOT FREEZE!**

NEED HELP?

CALL **855 MYLUNGS** TO SPEAK WITH YOUR PATIENT COACH



# REPEAT YOUR SAMPLE COLLECTION FOR 2 MORE DAYS FOR A TOTAL OF 3 DAYS

COLLECT YOUR SAMPLE IN THE **SAME CUP**

**REFRIGERATE** YOUR SAMPLE AFTER EACH COLLECTION



NEED HELP?

CALL **855 MYLUNGS** TO SPEAK WITH YOUR PATIENT COACH

# RETURN YOUR SAMPLE

NEED HELP?

CALL **855 MYLUNGS** TO SPEAK WITH YOUR PATIENT COACH

# RETURN YOUR SAMPLE ON YOUR LAST DAY OF COLLECTION

REMOVE YOUR SAMPLE FROM THE REFRIGERATOR

TIGHTLY SECURE THE LID

PLACE CUP IN BIO PROTECTIVE BAG

REMOVE FEDERAL EXPRESS PRE-ADDRESSED ENVELOPE  
FROM THE KIT BOX



*NEED HELP?*

*CALL 855 MYLUNGS TO SPEAK WITH YOUR PATIENT COACH*

# PREPARE TO SHIP YOUR SAMPLE TO THE LABORATORY

REMOVE COLD PACK FROM FREEZER AND PLACE IT IN THE KIT BOX

PUT YOUR SAMPLE IN KIT BOX NEXT TO COLD PACK

CLOSE THE KIT BOX CONTAINING YOUR SAMPLE BY FOLDING THE FLAPS INTO THE SIDES



NEED HELP?

CALL **855 MYLUNGS** TO SPEAK WITH YOUR PATIENT COACH

# SHIP YOUR SAMPLE FREE BY OVERNIGHT FEDERAL EXPRESS

PLACE THE KIT CONTAINING YOUR SAMPLE IN THE PRE ADDRESSSED  
FEDERAL EXPRESS ENVELOPE CONTAINED IN YOUR KIT BOX

**CALL 1-800-GOFEDEX (1800-463-3339) TO SCHEDULE A PICK UP**



NEED HELP?

CALL **855 MYLUNGS** TO SPEAK WITH YOUR PATIENT COACH



# PRECISION PATHOLOGY

*“A Spirit of Excellence”*



PRECISION PATHOLOGY HAS LICENSED THE INTELLECTUAL  
PROPERTY OF BIOAFFINITY TECHNOLOGIES, INC